UNIVERSAL SENSOR HOLDER

UNIVERSAL-SENSORHALTER

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PORTASENSORE UNIVERSALE

SUJETADOR DE SENSOR UNIVERSAL

DENTSPLY
RINN

559901 (2) 559921 (10) Anterior Fits Size 1, 1.5, 2	559906 (2) 559926 (10) Anterior – Thin Fits Size 1, 1.5, 2	559902 (2) 559922 (10) Posterior Fits Size 1, 1.5, 2	559904 (2), 559924 (10) Vert. Bitewing 559903 (2), 559923 (10) Hor. Bitewing Fit Size 1, 1.5, 2 (VBW can also be used for HBW with Size 0)	559905 (2) Endo Fits Size 1.5, 2 Horizontal Fits Size 1, 1.5 Vertical	559907 (2) Size 0 Periapical Fits Size 0 H or V	559913 (Short) 559914 (Long) Replacement Bands
--	---	---	---	---	---	--

Caution: U.S. Federal laws restrict this device to sale to or on the order of a dentist.

Indications for use:
XCP-DS FIT® holders are indicated for holding digital sensors for intraoral dental radiographs. XCP® and XCP-ORA® arms and rings are indicated for positioning and aligning intraoral dental radiographs.

Contraindications:
None known.

Warnings: Sterilize before use. DO NOT use phenol-based glutaraldehyde or ultrasonic cleaners. DO NOT chemically, dry-heat sterilize or cold sterilize. Only sterilize with steam autoclaving according to cleaning and sterilization instruction. It is the user's responsibility to qualify alternative methods of sterilization or disinfection. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination.

Precautions: Follow sterilization instructions carefully to prevent damage to plastic pieces during sterilization. Use sensor holders with a sensor cover. Do not pull on sensor cord during use.

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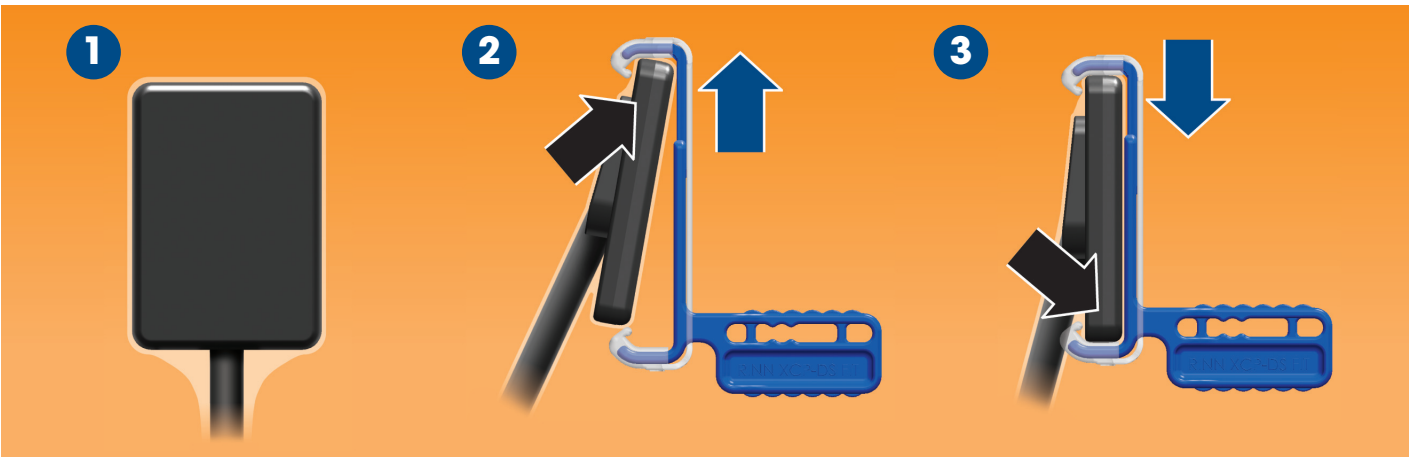
Step-by-Step Instructions:

See illustrations on reverse side.

- Place a protective cover over the sensor. **1**
- Load sensor into appropriate biteblock for the area to be imaged by holding sensor firmly and pressing edge of sensor against the movable clip. **2**
- Insert opposite sensor edge and release clip. **3**
- Insert positioning arm into the appropriate color-coded channel of the aiming ring and attach biteblock according to Positioning Guide.
- Correctly assembled, the sensor should be centered when viewed through the aiming ring.
- Position holder in patient's mouth and slide aiming ring close to patient's face.
- Align x-ray tube to aiming ring.
- Take exposure.
- Change as needed for the next desired image.

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Cleaning and Sterilization:

- Disassemble instrument(s). Separate holders, arm and ring for steam autoclaving.
- Remove debris from components with hot water and soap.
- Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms to avoid melting or warping.
- Cycle steam autoclave at 270° F (132° C) for 10 minutes at 30 PSI (206.8kPa). Do not exceed 273° F (134° C).
- To maintain sterility, instruments should remain bagged until ready for use.

Storage: Store at room temperature (18-25° C / 65-77°F).

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